



ULTRAFORMER III

Non-surgical Lifting, Tightening & Contouring System

Consultation Guide

Why Ultraformer III?

The Ultraformer III is a completely non-invasive, face lifting and tightening, body tightening and contouring device that provides a customized treatment procedure for the entire face, neck, and body.

Utilizing HIFU technology with Micro and Macro Focused cartridges, the Ultraformer III addresses signs of aging to restore youth and confidence of consumers around the world.

The Ultraformer III has also been approved to the standard of international regulatory bodies and associations which include Australia, Brazil, South Korea and Europe.

The Ultraformer III maintains a firm and growing presence in clinics and hospitals across more than 55 countries.



The Ultraformer Way

The Ultraformer III is easy to incorporate into clinics and may be used in conjunction with other treatment modalities. With no downtime, anesthetics, incisions, or side effects that affect your daily routine, the Ultraformer III treatment is the perfect “lunchtime” procedure.


The Ultraformer III provides:

- Quick and efficient treatments
- Micro & Macro focused cartridges
- Treatments for a wide demographic
- Solutions to address signs of aging




Skin Lifting for Age Shifting. Ultraformer III. Ultra-focused. Ultra-quick.

Comparable to reflecting sunlight through a magnifying glass, the Ultraformer III delivers ultrasound energy into the target region of the skin to stimulate collagen regeneration. The precision in delivery of a high peak power triggers the process of neo-collagenesis naturally within the body.

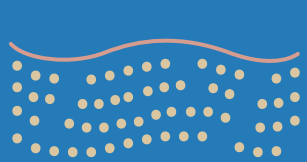


The dermis layer of the skin naturally synthesizes less collagen over the course of time.

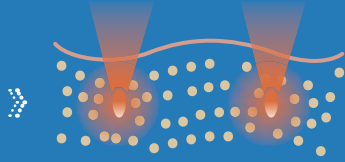


Focused ultrasound waves deliver energy directly within the skin layer causing a shrinking effect.



After inflammation and fibroplasia, new collagen is formed in the targeted area.

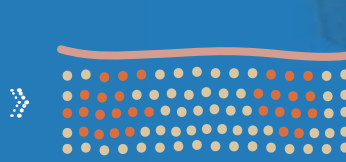


Through the regeneration of collagen, wrinkles are eliminated in the process of both firming and tightening of the skin.

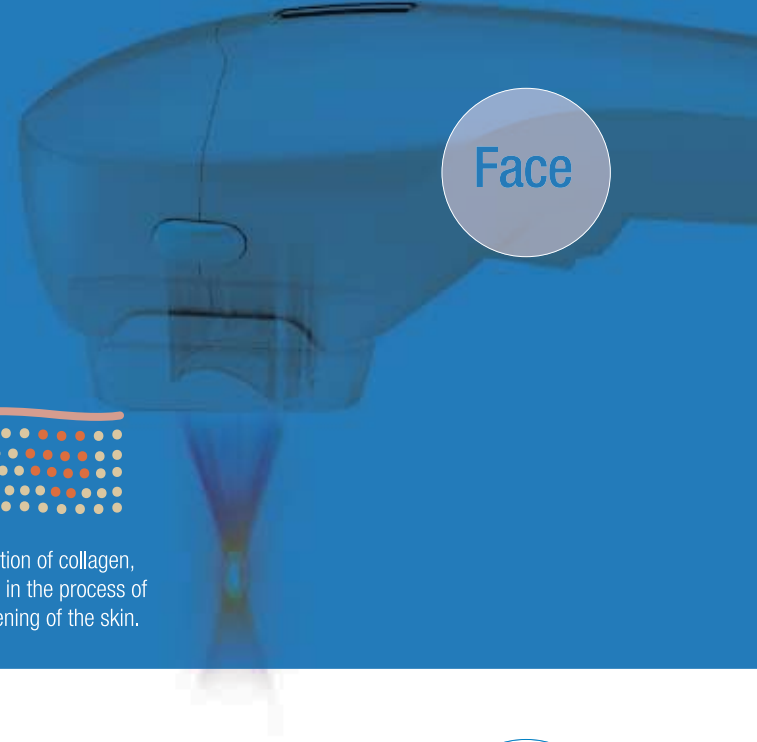


Eliminate stubborn fat cells in the easiest way possible. Non-invasive, no cutting, no stitches, no downtime.


Stubborn fat cells are difficult to burn off even through strenuous exercises and extensive weight loss regimens. The Ultraformer III treatment provides both non-invasive and innovative solutions without including side effects or damage to skin.

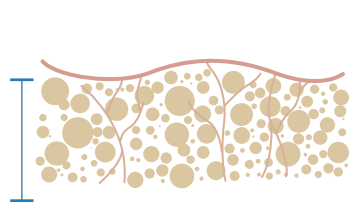


Fat cells and cellulite are formed within the body.

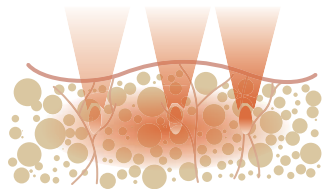

The Ultraformer energy destroys the fat cells that have been targeted.

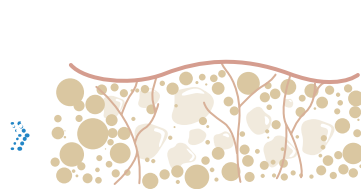

As a natural healing response of the body the cells regenerate, joining together to fill the gaps.

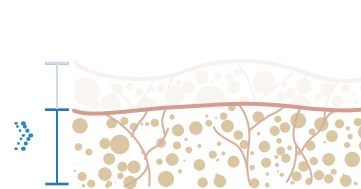

In result, the destroyed fat cells naturally exit the body to deliver a smooth contoured body line.





Forehead Wrinkles

Brow Lift

Crow's Feet

Fine Wrinkles
Frown Lines

Nasolabial Folds

Cheek Definition
Lip Lines

Marionette
Lines

Double Chin

Neck Folds

How does the Ultraformer III lift & tighten the face?

Multi-depth cartridges lift and tighten the face for a more youthful complexion. From the forehead down to the creases on your neck, the Ultraformer III provides the absolute solution to resolve complexes and apparent signs of aging.



1.5 mm



2.0 mm



3.0 mm



4.5 mm

How does the Ultraformer III tighten & contour the body?

The Ultraformer III provides competent tightening and contouring procedures to address loose or saggy skin and the contouring needs of patients. Multiple cartridges at various depths are utilized to target multiple layers of the skin to enhance skin elasticity quickly and efficiently.



6.0 mm



9.0 mm



13.0 mm



Before treatment

- Consult with your doctor or practitioner to assess whether you are suitable for treatment
- Consult with your doctor if you are currently taking medication
- If you have had prior treatment within the desired application area, consult with your practitioner or doctor

e.g fillers, botulinum toxins, or laser procedures



ON THE DAY of treatment

Face

- Wear comfortable clothing
- Remove make up
- Face treatment procedures vary between 20-30 minutes based on the targeted areas



ON THE DAY of treatment

Body


- Wear comfortable clothing
- Body treatment procedures vary between 30-40 minutes based on the targeted areas



After treatment

- No downtime necessary
- Makeup is applicable immediately following treatment
- The treated area may appear slightly red or pink. This will resolve within a few hours post treatment
- There are no post-procedure limitations on activity

**For Absolute
& Effective
Results**

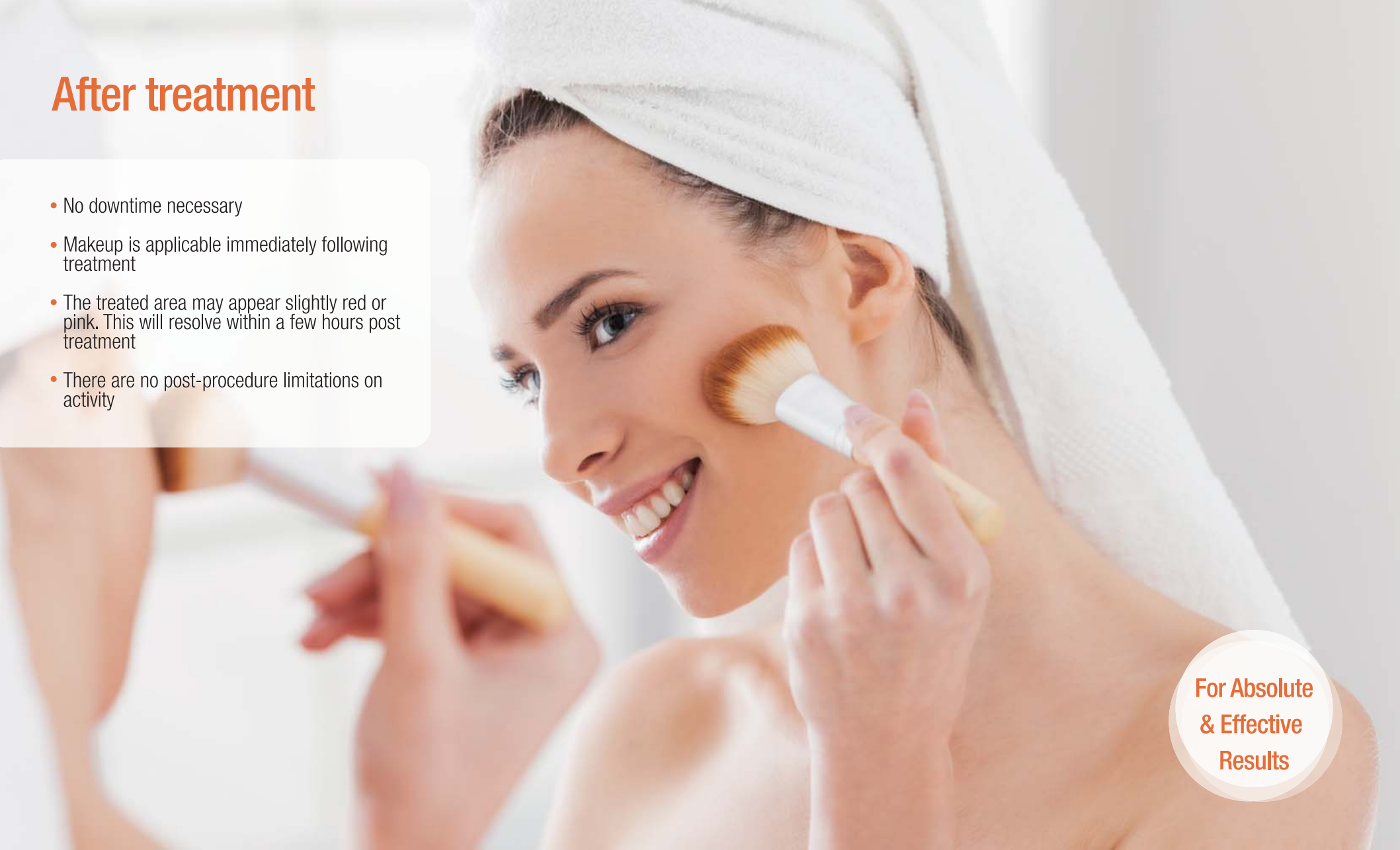

Clinical Cases

| Wrinkle Reduction & Cheek Definition




Baseline

Immediately Post Procedure

Procedure Information

Patient info Female / 31

Application

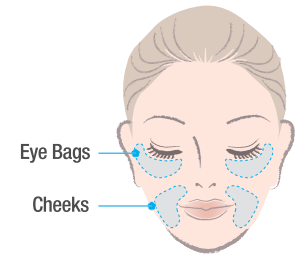

Result

Wrinkles visibly smoothed out, effective immediately after procedure.

Patient Comment

My fine lines were immediately less visible and it was such a simple procedure.

Clinical Cases

Wrinkle Reduction & Cheek Definition



Baseline




Immediately Post Procedure

Procedure Information

Patient info Female / 50

Application

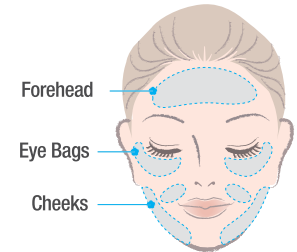

Result

Wrinkle lift visible.
Fine lines less defined.
Cheeks show a defined facial structure.

Patient Comment

There is more definition in my facial structure. I look younger already and I look forward to see what happens in the coming months!

Clinical Cases

| Brow Lift



Baseline

Immediately Post Procedure

Procedure Information

Patient info Female / 50

Application



Result

Uneven eyebrow treated.
Visible improvement, evening out
eyebrow positioning.

Patient Comment

My eyebrows were uneven before
the treatment but immediately
afterwards I can see the outstanding
difference.

Clinical Cases

| Double Chin




Baseline

9 Months Post Procedure

Procedure Information

Patient info Female / 42

Application

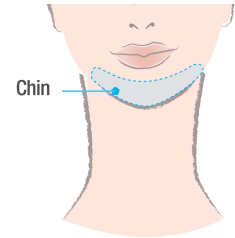

Result


Definition in the V-line of the chin is visible. Sagging form of the chin has been lifted.

Patient Comment

The sagging fat on my chin has tucked itself in. I can't wait to see the progressive improvement which is to come.

Clinical Cases

| Neck Folds

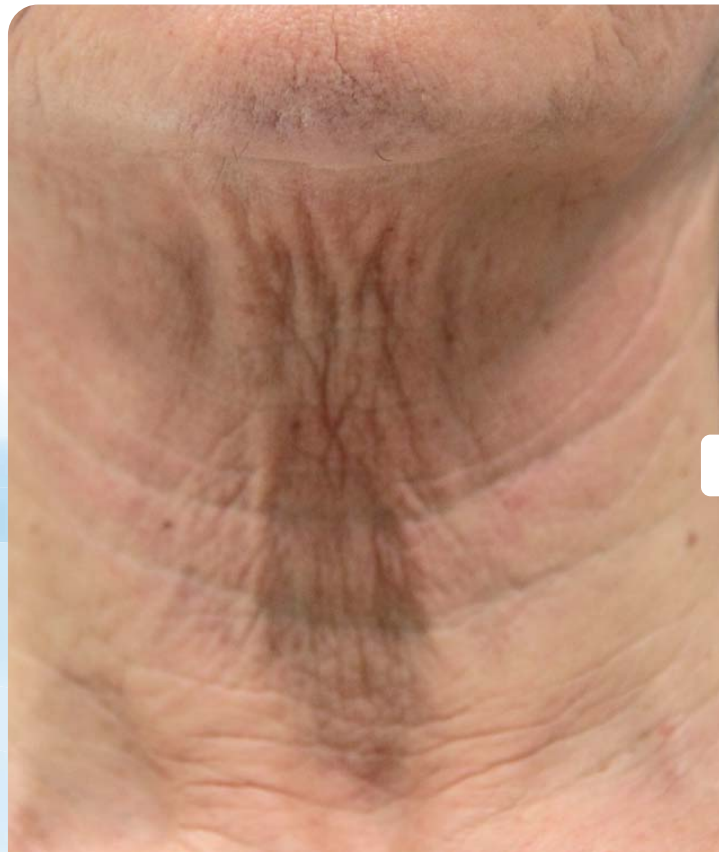

Baseline




3 Months Post Procedure

Procedure Information

Patient info Female / 50

Application

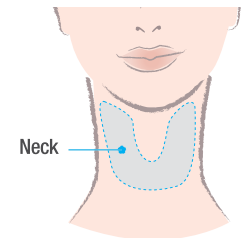

Result


Turkey neck has been stretched out. Exhibits a substantial improvement in wrinkle formation.

Patient Comment

My severe condition of turkey neck has disappeared. The results exceed my expectations immensely.

Clinical Cases


| Axillary Folds

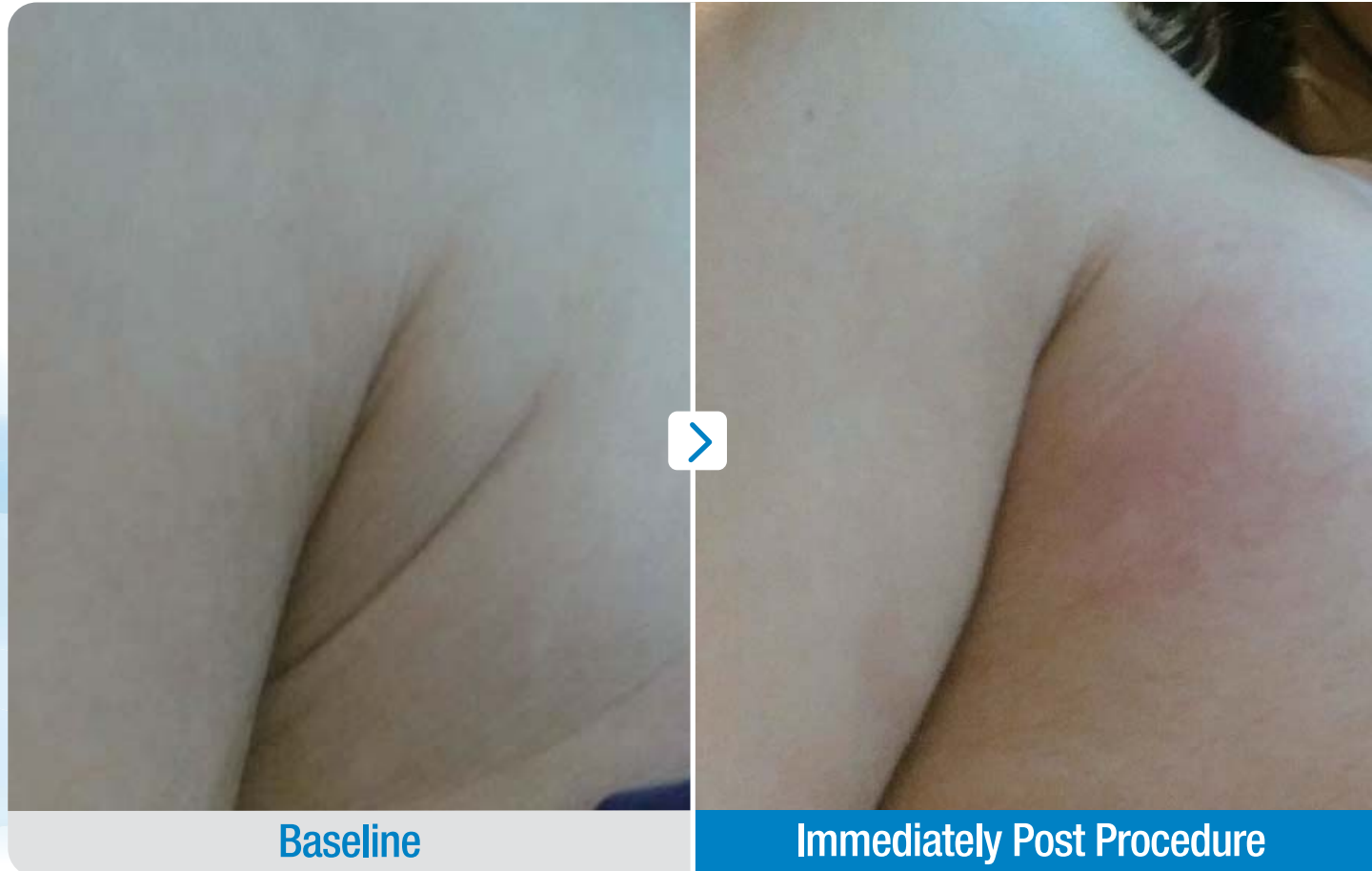

Procedure Information

Patient info Female / 30

Application

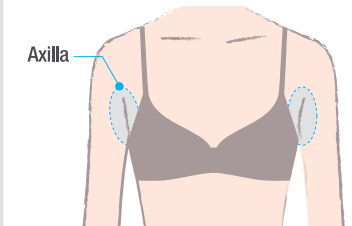

Result

Overlap of fat is not visible.
Good outcome.

Patient Comment

I can't believe the changes
that I'm seeing.
My rolls are not visible already?
The Ultraformer does not disappoint.

Clinical Cases


| Sagging Skin



Procedure Information

Patient info Female / 53

Application

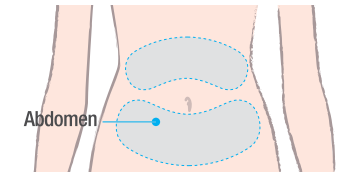

Result

Reduction of sagging skin.
General lifting of sagging skin.
Post 3 months of procedure.

Patient Comment

I didn't know the length and efficacy of results but after three months the results are outstanding.

FAQ

What are the advantages of the Ultraformer III?

The Ultraformer III utilizes cartridges focused on multiple depths of skin for lifting and tightening the face as well as tightening body tissues for contouring purposes. The patient can receive customized treatments on the face and body with the stable delivery of ultrasound energy, fast application, and precise accuracy of MMFU-powered transducers.

How is fat naturally removed from the body by the Ultraformer III?

Macro-focused depth cartridges damage fat cell membranes consisting of triglycerides and are then transported through the vascular and lymphatic systems to the liver. This process is made possible through the body's natural metabolism as the liver makes no distinction between fat disposed of by the Ultraformer III and fat originating from food consumption.

Are there any side effects?

Although both desired and competitive results for facial lifting and tightening and body contouring will likely show over the course of 3 months post treatment, patients may still experience slight redness or swelling for up to a few hours in the areas treated as well as temporary bruising which gradually subsides within 2-4 weeks.

“Uncovering your grin
from deep within”



FAQ

Is there any pain during treatment?

Many patients and practitioners report that in comparison with other devices, the Ultraformer III delivers nothing more painful than a sensation of mild discomfort.

Am I a candidate for the Ultraformer III?

Patients with diverse skin conditions such as fine lines, wrinkles, and hyperpigmentation on the face, or stubborn belly fat and love handles desiring a more curvaceous body figure, serve as ideal candidates for the Ultraformer III.

How long do the results last?

Patients may notice immediate results with progressive improvement up to 90 days post-treatment. These results can be maintained from several months to a year afterwards.



News & Magazines





For more information visit us
www.ultraformer.com
www.clatuu.com
www.classys.com
www.scizersonic.com

For further enquiries email us
info@classys.com

Reform your youth with a lunchtime procedure. Non-surgical Lifting, Tightening & Contouring System.

Classys products are available worldwide. For more information visit our website or email us at info@classys.com
The Ultraformer name and logo are registered trademarks of Classys Inc. Copyright © 2017. All rights reserved.

ULTRAFORMER III